



## VELscope Vx Imaging Adapter

### QUICK START GUIDE



- 1 Purchase an iPod touch 5<sup>th</sup> generation device from Apple Inc.
- 2 Follow the directions that come with the iPod to set it up with your Apple ID.
- 3 Go to the App Store and download the **VELscope Photo System** app.
- 4 Remove the Adapter Ring from the Adapter Bracket and attach it to VELscope Vx handpiece.



- 5 Remove the blue eyepiece from the VELscope Vx handpiece and, in its place, attach the Adapter Ring. To attach properly, align the Adapter Ring so that the “Unlock” symbol is at the top as shown below.



Then turn the Adapter Ring clockwise until it stops. When you are done, the “Lock” symbol on the ring should be at the top as shown below.



- 6 Snap the iPod touch into Adapter Bracket, insert the bottom of the iPod into the bottom of the bracket first and then push on the top of the iPod to snap it completely into the bracket.




- 7 The iPod can then be attached to the VELscope Vx handpiece in either horizontal or vertical orientation as shown below.



The Adapter Ring and Bracket are magnetic so that the iPod can be attached and removed conveniently by inserting the metal disks on the bracket to attach and just pulling off to remove.

- 8 Launch VELscope Photos System app and follow the **Help** link on the app title screen to link to the VELscope website for detailed guidance and instructions.

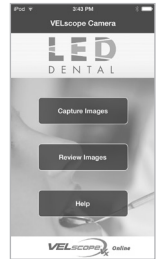

### LED Dental Inc

580 Hornby St. Suite 810  
Vancouver, BC, V6C 3B6 Canada  
North America Toll Free : +1-888-541-4614  
Phone: +1-604-434-4614

Apple and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries.  
© Copyright 2015, LED Dental Inc.  
All rights reserved. LED 0291 Rev A.