


Initial Setup

Caution: DO NOT look into the patient-side of the handpiece or shine the device into anyone's eyes.

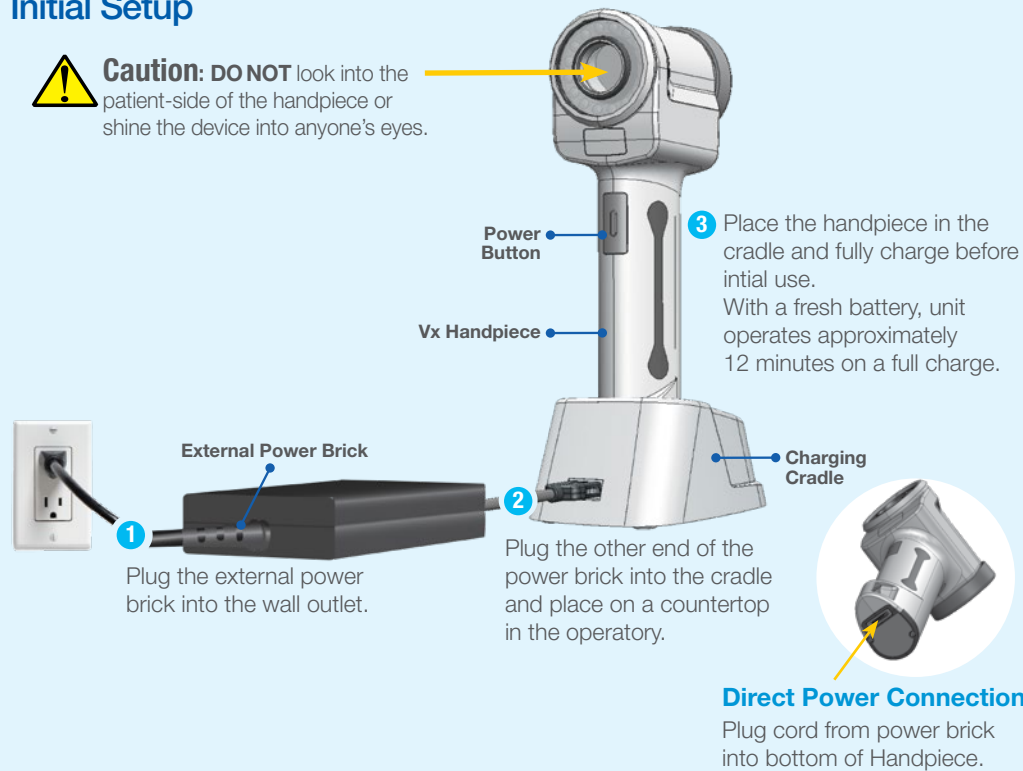

Disposables

For safety and performance reasons, a new VELcap Vx and VELsheath Vx should be used for each patient.
PLEASE DISPOSE OF THESE ITEMS AFTER A SINGLE USE.

Follow these steps prior to every examination



Place a VELsheath Vx over the entire handpiece.



Attach a VELcap Vx to the handpiece.

NOTE: Order additional disposables from your dealer or at velscope.com



Quick Reference Guide

System Components

- VELscope Vx Handpiece (Battery Included)
- Charging Cradle
- External Power Brick and Power Cord
- Patient Safety Glasses
- Operation Manual
- Quick Reference Guide
- Step-by-Step Examination Guide
- Sample Disposables
- Patient Brochures

Register your VELscope Vx today at velscope.com

Registration provides:

- Warranty verification
- A free 1-hour training webinar
- Inclusion in our online listing VELscope practice database so patients near you who want a VELscope exam will know that your office offers them.

velscope.com is a valuable resource

- Product News
- Company Updates
- List of tradeshow & events
- Useful support contacts
- Education and training materials

Customer Support:

support@velscope.com

General Information:

info@velscope.com

Sales Information:

sales@velscope.com

Webinar Information:

webinars@velscope.com

LED Dental Inc

580 Hornby St. Suite 780
Vancouver, BC, V6C 3B6 Canada

North American Toll Free: +1 888 541 4614

Tel: +1 604 434 4614 Fax: +1 604 434 4612

EC REP

Obelis s.a
Bd. Général Wahis 53
1030 Brussels, BELGIUM
Tel: +(32) 2.732.59.54
Fax: +(32) 2.732.60.03
E-Mail: mail@obelis.net

Quick Reference Guide



- The Vx Handpiece may get warm – this is normal as the device is a high power instrument.
- Maintain the front of the handpiece at least 3” from the tissue to ensure uniform illumination.
- Ensure patient wears Patient Safety Glasses.
- To maximize battery life, place Vx Handpiece back in the Charging Cradle after use.

Additional Notes

- The Vx Handpiece is a delicate instrument, do not drop.
- Darken the room as much as practicable to minimize ambient light (turn off dental operatory light and if possible dim overhead lights, close blinds, face patient away from direct light etc.)

Cleaning the VELscope Vx handpiece

Wipe down with a hospital-grade surface disinfectant and a towelette or gauze, e.g. Caviwipes™ or equivalent. Use a soft tissue or lens cloth to clean the glass filters if required.


Caution:

DO NOT immerse in any liquid or use excessive cleaning fluid when cleaning the system. The Vx Handpiece and Charging Cradle are not water resistant and excess fluid can penetrate the system and cause damage to internal components. This damage is not covered under the warranty.

Warning:

Ensure that liquid does not pool down in the Charging Cradle near the electrical connector. Any residual liquid present near the connector in the Charging Cradle may cause an electrical short and a possible fire hazard.

Vx Handpiece Status Indicators

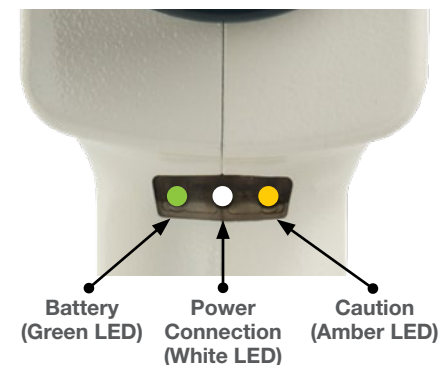

Battery Green	Power Connection White	Caution Amber	Device Status
Normal Operation Cordless			
🔴 Flashing	⦿ Off		BLUE Light ON Recharge Battery Soon
⦿ Off	⦿ Off		BLUE Light NOT Functioning Recharge Battery
Normal Operation Connected Directly to External Power Brick or in Charging Cradle			
🔴 Flashing	● Solid		Charging
● Solid	● Solid		Fully Charged
Problem Detected			
		🔴 Flashing Duration: 10 Seconds	Blue Light Shuts Off Wait 1 minute – if problem persists, see Operation Manual